


433-CMU-40 Economy Central Monitor

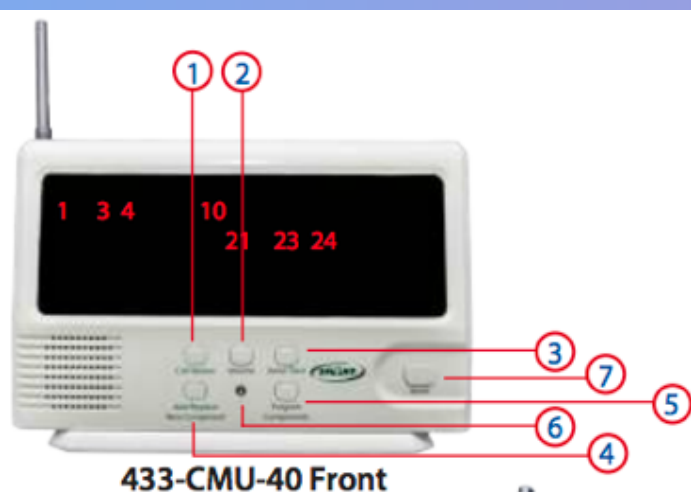


- 1) Call Review Button. 2) Volume Button. 3) Voice/Tone Button. 4) Add/Replace New Component Button. 5) Programme Components Button. 6) Programming Light. 7) Reset Button. 8) Antenna. 9) Hook. 10) Battery Compartment. 11) Base. 12) Call/Alarm Switch. 13) Silencer Port. 14) Lock/Unlock Switch. 15) AC Adaptor Input

Programming / Setup

- ◆ Insert 4x "C" size batteries in rear battery compartment (10) (remove screw on battery cover to access) or plug in AC adaptor.
- ◆ Put the 433-CMU into the unlock position (14)
- ◆ To Select voice or tone press the voice/tone button (3) repeatedly to scroll through the options. Stop at desired selection
- ◆ Adjust volume button (2) to desired volume.
- ◆ Place components in order you want to programme them (pads should be on a flat surface and all items within reach)
- ◆ Press and hold the add/replace button (4) for one beep.
- ◆ Next press program components button on the 433-CMU (5) for two beeps plus closing melodic tone. Programming light will be on
- ◆ Flashing number on screen is the last number a component is currently programmed to. For new 433-CMU's the screen will be blank.
- ◆ Activate first component (press & release nurse call button **or** turn motion sensor on and off **or** press bed/chair pad or turn transmitter on for floor mat **or** touch the two pasts of the window/exit alarm together and then seperate)
- ◆ Press reset button twice
- ◆ Press Add/Replace button for two beeps plus closing melodic tone
- ◆ Next press programme components button for one beep. Programme light will be steady and ready for next component to be programmed.
- ◆ Repeat 2 **red** bullet point steps above for each component added
- ◆ Put 433-CMU into lock position (14)
- ◆ Press reset button (7) & test components

Notes

- ◆ For use with up to 40 cordless bed pads, chair pads, floor mats, motion sensors, nurse call buttons, reset buttons , window/ door exit alarm
- ◆ For use with the The 433-PGD Pager. No programming of pager is required, it will sync with the 433-CMU.
- ◆ **Programming must be done quickly – if no activity when in programinbg mode, unit will exit programming mode automatically after 20 seconds**
- ◆ Bed Pads and Chair Pads will have a 3 second delay when activated
- ◆ Bed Pads and Chair Pads will reset themselves if pressure is re-applied.

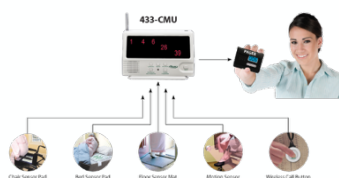

433-PGD Pager (optional extra)